



FRED easy



CARDIAC
defibrillators



**The Automated External Defibrillator
that accompanies all first aid rescuers in
their life-saving mission**



SCHILLER

The Art of Diagnostics

FRED easy

The FRED easy Automated External Defibrillator is the best life-saving tool for all emergency services: paramedics, ambulances, firefighters....

It is light, rugged and user friendly and therefore adapted to all situations.


FRED easy features the unique pulsed biphasic waveform MULTIPULSE BIOWAVE with patient impedance compensation. This waveform is efficient for adults and children thanks to the automatic recognition of children pads and energy adaptation.

Adapted voice and displayed instructions for first aid rescuers together with a fast rhythm analysis and shock delivery guide the rescuer during the resuscitation process and minimise the hands-off time.

In order to follow the patient throughout the resuscitation process, the intervention data is recorded and can be reviewed on a PC and a report can be generated.

FRED easy is available in a range of versions and options to fulfil the rescuer's needs:

- Semi or fully automatic
- Metronome: providing the correct rhythm for chest compressions
- ECG display
- Manual override: doctors can switch off the AED mode and decide whether they want to defibrillate or not

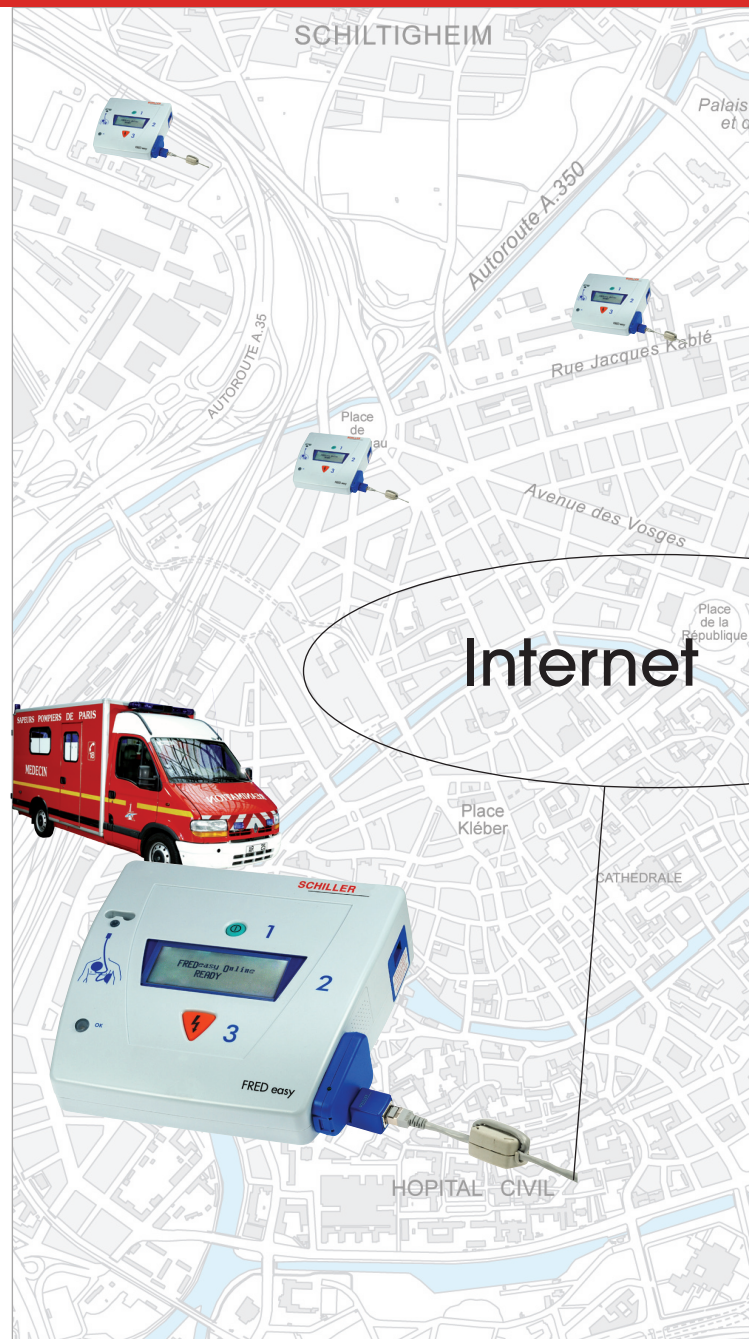


FRED easy with manual override



FRED easy fully automatic



ECG display

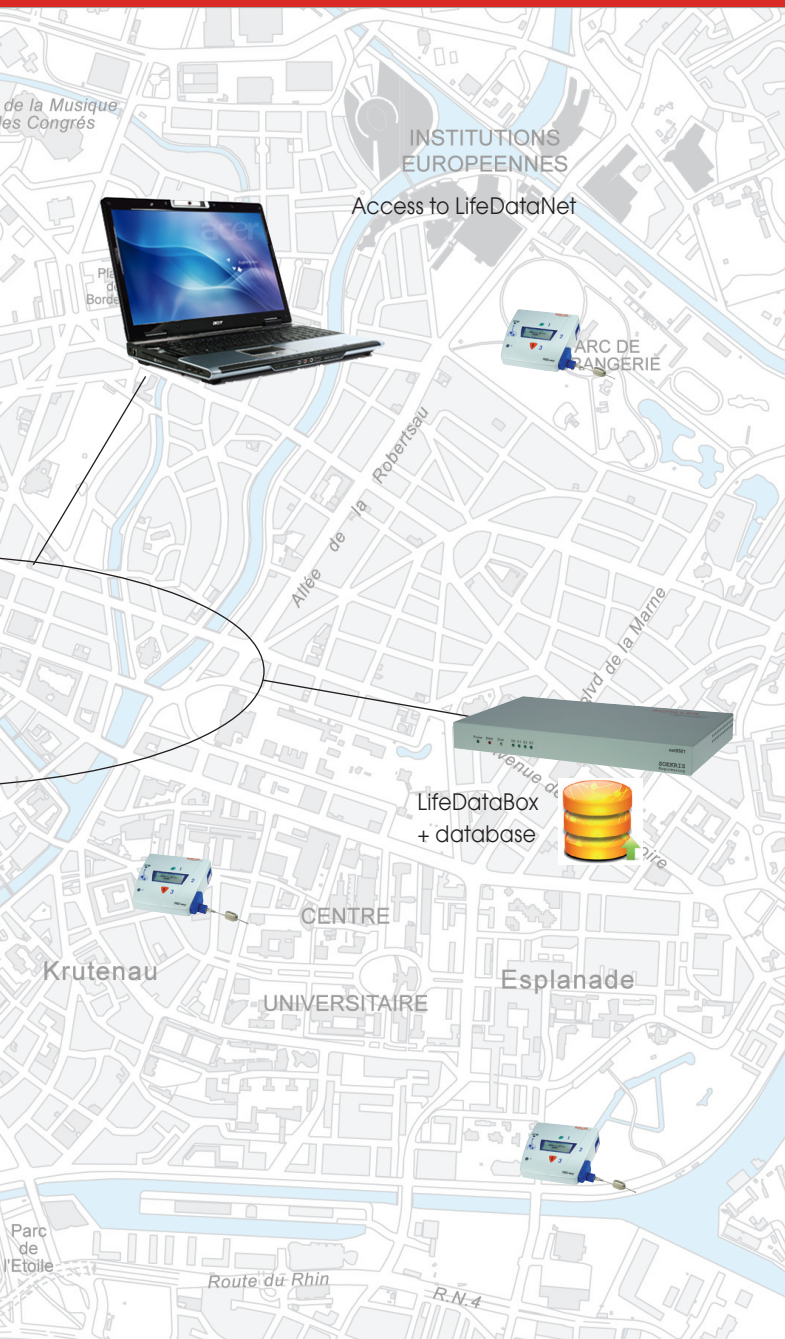

FRED easy Online

The first defibrillator featuring automatic data transmission. Intervention data and the device configuration can be sent to a server via an Ethernet network.

Advantages for everyone:

Transmission is easy, fast and safe. The data is stored centrally for direct and immediate access by authorised persons from any computer connected to the network. Multi-user access to the data via any computer connected to the network (hospital, treating physician, cardiologist, rescue services, administration). This system considerably simplifies the handling of medical and administrative patient data (no paper, no memory card).

LifeDataNet:

The transmitted data is stored on a central server, LifeDataBox, with the LifeDataNet application.

Two types of data can be transmitted:

- Patient data (events, ECG curve, patient impedance)
- Configuration and maintenance information

The LifeDataNet application is a simple and user-friendly interface enabling you to:

- see which device has transmitted the data (serial number, identification, transmission date and time)
- download and analyse patient data
- view the device settings (mode, energy settings, options)
- view the maintenance status (self-test result, battery status, software version).

When a first aid vehicle comes back to the station, simply connect the FRED easy to the network and the data is automatically downloaded and the device ready for a new intervention.

Schiller also offers ARGUS PRO LifeCare 2 and DG4000 to perform advanced patient resuscitation and monitoring.



ARGUS PRO LifeCare 2



DEFIGARD 4000



CARDIAC
defibrillators



Asia

SCHILLER Asia-Pacific / Malaysia
52200 Kuala Lumpur, Malaysia
Phone +603 6272 3033
Fax +603 6272 2030
sales@schiller.com.my
www.schiller-asia.com



Austria

SCHILLER Handelsgesellschaft m.b.H.
A-4040 Linz
Phone +43 732 709 90
Fax +43 732 757 000
sales@schiller.at
www.schiller.at



China

SCHILLER Int'l Trading (Shanghai) Co. Ltd.
200335 Shanghai, China
Phone +86-21-62099627
Fax + 86-21-62099623
sales@schiller.com.my
www.schiller.cn



France

SCHILLER Médical S.A.S.
F-67162 Wissembourg/Cedex
Phone +33 3 88 63 36 00
Fax +33 3 88 94 12 82
info@schiller.fr
www.schiller-medical.com



France (distribution France)

SCHILLER France S.A.S.
F-77600 Bussy St Georges
Phone +33 1 64 66 50 00
Fax +33 1 64 66 50 10
infoschiller@schiller-france.fr
www.schiller-france.com



Germany

SCHILLER Medizintechnik GmbH
D-85521 Ottobrunn
Phone +49 89 62 99 81-0
Fax +49 89 609 50 90
info@schillermed.de
www.schillermed.de



Hungary

SCHILLER Diamed Ltd.
H-1141 Budapest
Phone +36 (1) 383-4780 / 460-9491
Fax +36 (1) 383-4778
sales@schiller.at
www.schiller-hungary.hu



India

SCHILLER Healthcare India Pvt. Ltd.
Mumbai - 400 001, India
Phone +91 22 6152 3333/ 2920 9141
Fax +91 22 2920 9142
sales@schillerindia.com
www.schillerindia.com



Japan

SCHILLER Japan, Ltd.
Hiroshima 734-8551
Phone +81 82 250 2055
Fax +81 82 253 1713
koji.maekawa@schiller.jp
www.schiller.jp



Poland

SCHILLER Poland Sp. z o.o.
PL-02-729 Warszawa
Phone +48 22 8432089
Fax +48 22 8432089
schiller@schiller.pl
www.schiller.pl



Russia & C.I.S.

SCHILLER AG Rep. office
125124 Moscow, Russia
Phone +7 (495) 970 11 33
Fax +7 (495) 970 11 33
mail@schiller-ag.com
www.schiller-cis.com



Spain

SCHILLER ESPAÑA, S.A.
E-28230-Las Rozas/Madrid
Phone +34 91 713 01 76
Fax +34 91 355 79 33
schiller@schiller.es
www.schiller.es



Switzerland

SCHILLER-Reomed AG
CH-8953 Dietikon
Phone +41 44 744 30 00
Fax +41 44 740 37 10
sales@schiller-reomed.ch
www.schiller-reomed.ch




Turkey

SCHILLER TÜRKİYE
Okmeydanı-Sisli - İstanbul
Phone +90 212 210 8681 (pbx)
Fax +90 212 210 8684
sales@schiller-turkiye.com
www.schiller-turkiye.com



USA

SCHILLER America Inc.
Doral, Florida 33172
Phone +1 786 845 0620
Fax +1 786 845 06 02
sales@schilleramerica.com
www.schilleramerica.com

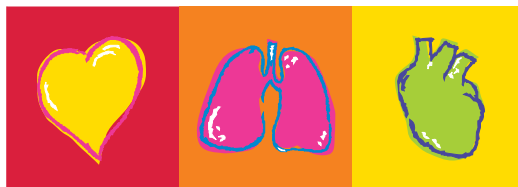

SCHILLER

The Art of Diagnostics

11/11

Part no.: 2.500378